Stethoskope

Gebrauchsanweisung | User Instructions | Mode d'emploi
Istruzioni per l'uso | Instrucciones de uso

DE

EN

FR

IT

ES

Symbole

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Hersteller

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Bestellnummer

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Importeur | Händler

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Batch designation

Unique device identification

European Authorized Representative

Contents of package

- Stethoscope
- Spare eartips
- Spare diaphragms
- User instructions

General information

- Your bosco stethoscope complies with the ordinance (EU) 2017/745 on medical devices (CE mark).

Please read these user instructions carefully before using the stethoscope and store them afterwards.

 Never expose your stethoscope to extreme cold or heat.

 Do not perform heat sterilisation on the stethoscope.

 The stethoscope can be damaged by sharp or pointed objects.

 Do not insert the eartips too deep into the auditory canal – risk of injury!

 Check the eartips for soiling and remove it to ensure proper function of the stethoscope.

 Never use the stethoscope without eartips – risk of injury!

 Never use the stethoscope without the diaphragm.

 Use only original bosco spare parts.

 The stethoscope may only be used for the purpose indicated under “Intended use”.

 If there are signs of material wear, the device should no longer be used.

Intended use

Assessment of sounds and noises in human medicine associated with the activity of the heart, lungs, and intestines. During blood pressure measurement, a stethoscope is used to hear Korotkoff sounds.

EN

Indication

A stethoscope is used in human medicine in general to capture and assess sounds and noises associated with the activity of the heart, lungs, and intestines.

During blood pressure measurement, the stethoscope is used to hear Korotkoff sounds. The device is thus suitable for the supplementary diagnosis of cardiovascular diseases, diseases of the respiratory tract, and diseases of the gastrointestinal tract.

Contraindication

No contraindications are known.

Target patient group

The boso stethoscopes are suitable for auscultation in adults, children, infants, and newborns.

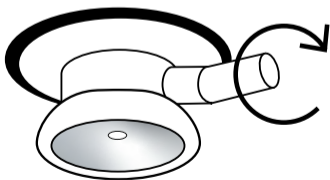
Intended users

The boso stethoscopes are used by physicians and medical professionals.

EN


Using the double-sided chest piece (only boli, boki, cardio)

- By rotating the chest piece around the connection, the stethoscope is adjusted for use of the diaphragm side or the bell side. Rotate until you clearly feel the piece snap into place.
- Use the diaphragm side for the auscultation of high frequencies and the bell side for the auscultation of low frequencies.
- With the binaural on, you can immediately hear whether the diaphragm is active by lightly rubbing the diaphragm.

Adjusting the binaural

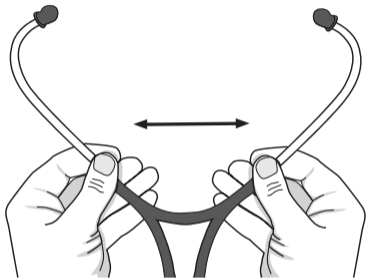
- Your boso stethoscope has been constructed and adjusted in the factory so that it adapts comfortably to and acoustically seals the ear.
- When inserting the eartips into the ears, they should point forward in the direction of the auditory canal.
- If the spring tension is nonetheless too high or too low for you personally, it can be easily adjusted:

Too high:

Hold the binaural firmly as shown and slowly pull the two sides apart until the binaural is sufficiently expanded.

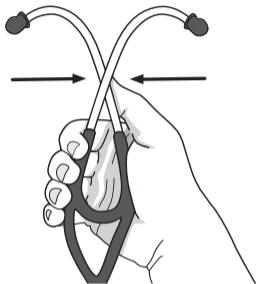Adjusting the binaural

EN

Repeat the procedure until the desired tension is reached.

Too low:

Hold the binaural in your hand at the connector between the metal and plastic and slowly squeeze it together until both plastic tubes cross over each other.

Changing the diaphragm

- To remove the diaphragm, hold the chest piece with the diaphragm side facing upwards and roll the protective ring off downwards using both thumbs and index fingers.
- To mount the diaphragm, lay it on the chest piece and place the protective ring with the shorter edge facing downwards directly on the diaphragm. Secure the protective ring at one position using your thumb and index finger. Now push the protective ring down with your other thumb, using circular motions along the ring. Align the protective ring such that the diaphragm is fully secured.

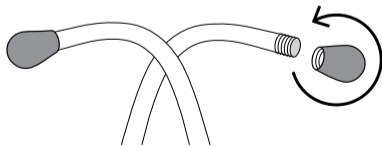
Changing the eartips

- To change the eartips, unscrew the tips over the integrated thread and affix the new tips by screwing them on.

EN

To avoid injuries, check that the tips are correctly positioned prior to each use.

Cleaning

For cleaning, use a cloth dampened with a mild soap solution or alcohol (70% isopropyl).

Never use abrasive cleaning agents, thinners, or benzene for cleaning.

Do not place the stethoscope in cleaning fluid.

Disinfection

We recommend using mikrozid[®] sensitive liquid disinfectant (Schülke & Mayr) to disinfect the device by wiping it, leaving the disinfectant on for at least 5 minutes.

Incident reporting obligation

A serious incident is to be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is located.

A “serious incident” denotes an incident which had, could have had, or could have one of the following consequences, directly or indirectly:

a) the death of a patient, user, or another person,

b) the temporary or permanent serious worsening in the state of health of a patient, user, or other persons,

c) a serious risk to public health.

Warranty conditions

EN

We provide a three-year manufacturer's warranty on this product from the date of purchase. The date of purchase must be shown by the invoice. Defects due to material or manufacturing errors are repaired free of charge within the warranty period. If parts are replaced under warranty, the warranty period is extended only for those parts, not for the entire device.

The warranty does not cover damage due to wear and tear, transport damage or any damage caused by incorrect use (e.g. failure to comply with the user instructions) or by interventions by

unauthorised parties. The warranty does not give you any claim for damages against us.

This does not affect the purchaser's right to make a claim on the grounds of defects as set out in § 437 of the German Civil Code.

If you wish to make use of the warranty, you must send the device together with the original receipt to:

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BOSCH + SOHN GmbH u. Co. KG
Ningbo Free Trade Zone Tenso Medical Instruments Co., Ltd.
Shanghai International Holding Corp. GmbH (Europe)